

## DERMAPEN IS RECOMMENDED TO TREAT:

- Aged sagging skin
- Fine lines & wrinkles
- Smokers lines around the lips
- Skin tightening and rejuvenation
- Large pores
- Stretch marks
- Scars (acne, surgical and burns)
- Alopecia\*
- And much much more..

\*when used in conjunction with topical treatment

# B&A

CLINICAL IMPROVEMENT IN BURN WOUND & SCAR TISSUE  
(8 SESSIONS, ONCE EVERY 2 WEEKS) <sup>1</sup>



IMPROVES ACNE SCARS & MINIMIZES PORE SIZE  
(5 SESSIONS, ONCE EVERY 2 WEEKS) <sup>1</sup>



\* Individual results may vary

Make an appointment with  
your clinician today

Or for more information contact  
[dianner@genop.co.za](mailto:dianner@genop.co.za)

Check with your doctor before proceeding with Dermapen procedure if you are: Pregnant, using blood thinners such as warfarin or if you are susceptible to keloiding skin. Do not treat over keloid scars, active problematic skin such as: Severe solar keratosis, raised moles/warts, Rosacea, Eczema or Psoriasis.

**Genop**  
healthcare

Reference: 1. Equipped data on file.  
Genop Healthcare (Pty) Ltd. (Co. Reg. no. 1984/011575/07).  
72 Morkels Close, Capital Hill Commercial Estate, Cnr of K101 and Le Roux Road,  
MIDRAND. © 2012 Tel: 0861 436 674 [www.genop.co.za](http://www.genop.co.za) 10/2012/TC/130.  
10/12 12416

**YOUR GUIDE  
TO SKIN NEEDLING  
THERAPY WITH**  
**Dermapen**  
Advanced skin needling



## WHY CHOOSE


**Dermapen**  
Advanced skin needling

Dermapen's advanced skin needling technology uses multiple needles that vertically pierce the skin with an automatic vibrating function. When moved across the skin's surface it stimulates natural collagen reproduction, whilst allowing fine channels to be created in the skin. The channels can carry up to 80% more topical nutrients, allowing for treatment delivery deep into the skin's fibroblasts to feed the underlying cells found in the dermis and basal layer. This increased absorption of active ingredients into the skin creates stimulated repair and faster resurfacing results.

Advanced skin needling promotes scar-less healing and the deposition of normal woven collagen, similar to fractional laser treatments but without the side effects and downtime. It's also more effective than most microdermabrasion and chemical peels.

Unlike traditional skin needling rollers, the Dermapen causes less trauma and has minimal epidermal (skin) damage, therefore reducing pain and discomfort, and making it a safer more effective treatment. The disposable needles eliminate risk of cross-contamination, whilst the adjustable needle depths allow for a treatment tailored to your individual needs. The Dermapen is so advanced, that it is even suitable and safe to use in delicate and hard to reach areas such as around the eyes and lips and on the nose.

Your clinician will assess your skin and treatment area to recommend a treatment plan suitable for you.

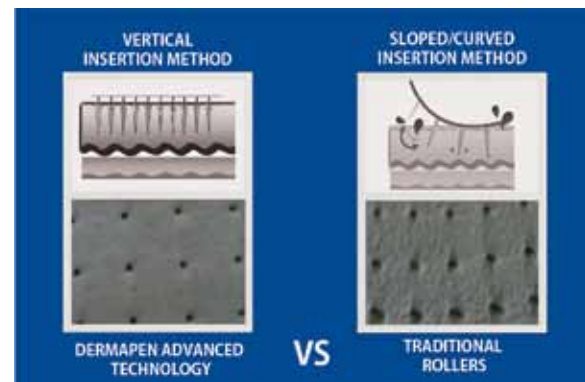

# FAQ's

## Q How many treatments are recommended for rejuvenation, acne scarring and stretch marks? (General guide)

A Rejuvenation:	4-6 treatments	4 weeks apart
Acne scarring:	6 treatments	2 weeks apart
Stretch marks:	<>10 treatments	3 weeks apart

## Q Should I prepare my skin before having a DermaPen Treatment?

A For optimal results we recommend a period of effective active treatment creams such as NeoStrata Skin Active Antioxidant Defense Serum to allow the skin's strength to build-up (especially for thin or delicate skin areas). Continue to use these recommended solutions after the treatments for a period of 6 weeks.

## Q How long before I see results?

A Typically, you will see results after the first treatment. Lasting and more significant results will occur after 4 to 6 treatments (spaced 2-4 weeks apart). Your skin will continue to improve over the next 6-12 months after a course of treatments and when combined with the recommended post treatment care.

## Q Is the treatment painful?

A For minimum needle depths Dermapen treatments are painless. Treatments over 1mm deep, a topical anaesthesia cream is applied for 15-40 minutes before treatment and will render the treatments completely pain-free.

## Q Is there any downtime after treatments?

A Generally no, but you may experience facial redness for 12-48 hours after the treatment. Most people are able to return to normal daily activities immediately after the treatment. Application of a breathable camouflage will seal the skin and cover any redness for an immediate return to work.

## Q How soon after filler injections can I have a DermaPen treatment?

A It's recommended to leave at least 4 days after filler injections so that any post treatment bruising from the fillers can settle. Or have your Dermapen treatment 1-2 days before having fillers injected.

## HOW TO CARE FOR YOUR SKIN BETWEEN TREATMENTS

### STEP 1.

#### Advanced Resurfacing

Your clinician will apply a suitable topical treatment to compliment your advanced skin needling. Between treatments continue using this recommended treatment solution (refer to the product information for application instructions).

### STEP 2.

#### Hydrate and Strengthen

For best results, it's important to keep your skin hydrated and protected from the harsh environment to make it healthy and strong. We recommend a night and day treatment cream, used daily over a period of 90 days. If you'll be exposed to the sun always wear appropriate sun protection.

### STEP 3.

#### Cover, Recover and Protect for Beautiful Skin

It's unlikely you'll have any side effects after your treatment. However, if you do experience redness or sensitive skin we recommend you use a post procedure, breathable makeup. This will help you get back to your everyday activities without anyone suspecting you've even had a treatment. Plus with makeup camouflage coverage it can work as sun protection for the skin.